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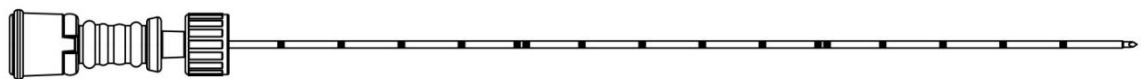
ENGLISH**Intended use:**

The products are intended for soft tissue biopsy in conjunction with the biopsy needles used for taking samples.

Contraindication:

- Please note the contraindications for histological tissue removal.

Warnings:

- Consult the product table when choosing the sheaths and corresponding biopsy needles.
- The PUNCTURE SHEATH needle is packaged in a single bag of medical paper and transparent laminate and is sterilized with Ethylene Oxide (EO) in compliance with the requirements of the ISO 11135 standard. It is marketed in packs of 10 pieces per box. The product is sterile. Do not use the product if the packaging is damaged or if the shelf-life of the product has expired.
- Aseptic techniques must be used in a sterile environment.
- The products are NOT designed to be used in the MR area!

Precautions for use:

- This product may be used by physicians only.
- This instrument has been specifically manufactured for single use only. The product may not be re-used after a single use. The quality of materials, coatings, and adhesive joints may deteriorate. Safe use is no longer guaranteed. The product is not designed for cleaning and sterilization processes that are necessary after a single use. The sterility of reprocessed single-use products is therefore not guaranteed. The risk of unintentional injuries and infections, especially cross-infections between patient and medical personnel, increases disproportionately. The company Somatex does not accept any liability in the event of resterilization and reuse of the instrument.
- Carefully inspect the product to ensure that it is not damaged or bent.
- Handle with care. It is advised NOT to use the product if the needle will be subject to high torsion and tensile forces, under which the needle or the handle might break.

Notes:

- Please consult the instructions for use of the biopsy needles to be used.
- Use imaging techniques (sonography, tomography) for application.

Application:

1. Check the expiration date.
2. Check that the diameter and length of the product are compatible with the biopsy needle to be used. Note: The biopsy needle is not included with this product.
3. Disinfect the skin, perform local anesthesia, and, if necessary, make a stab incision.
4. Working under aseptic conditions, carefully remove the product from the packaging.
5. Puncture the skin and guide the needle to the desired biopsy site.
6. Use imaging to confirm the correct position.
7. Remove the stylet.
8. Extract the sample with the biopsy needle.
9. After the sample has been extracted, gently pull the needle out of the body and treat the wound.

Disposal: Dispose of the product according to internal guidelines and applicable regulations.

Storage: Store in a dark, dry location. Protect from heat and direct sunlight.

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SOMATEX Medical Technologies GmbH